

My ilo Curve Instructions

Instructions

1. Clean the Curve before use.
2. Press and hold the on/off button to turn the Curve on. The indicator light will flash 3 times. Immediately press the on/off button again to change the vibrating patterns, 8 vibration patterns will cycle.
3. Press the triangle button while vibrating, the first press will reset the device to the first vibration pattern, click again to pause vibrations. Press and hold the triangle button to increase the vibrating frequencies. The faster the frequencies are, the faster the indicator light will flash. The indicator light will stop flashing when you have reached the maximum frequency.
4. Press the on/off button for 3 consecutive seconds to turn off.
5. Wash the Curve after use.

Charging time – 1.5 hours

Usage time – 3 hours

Caution: Do not charge when the power is switched on. Wash product before and after use. If you feel uncomfortable while using this product, please stop using immediately and seek medical treatment if necessary.

Check out our Youtube channel for a short video of how this product works – [Restorative Sexual Health Clinic - YouTube](#)