

My ilo Egg Instructions

Instructions

1. Clean the Egg before use.
2. Press and hold the on/off button for 3 consecutive seconds to turn the Egg on. Click the on/off button to change vibration patterns, 10 vibrating patterns will cycle. Massage and stimulate.
3. Press the on/off button for 3 consecutive seconds to turn off.
4. Wash the Egg after use.

Charging time – 1 hour

Usage time – 1.5 hours

Caution: Do not charge when the power is switched on. Wash before and after use. If you feel uncomfortable while using this product, please stop using immediately and seek medical treatment if necessary.

Check out our Youtube channel for a short video of how this product works –
[Restorative Sexual Health Clinic - YouTube](#)